

Suction Bathroom Grab Rail

IMPORTANT SAFETY INSTRUCTIONS

Use only after thoroughly reading instructions.

Incorrect use can result in serious personal injury.

PRECAUTIONS:

The Suction Bathroom Grab Rail is not designed to hold or support total body weight. It is only designed to assist with your natural motion whilst moving around the bathroom.

- o Always be aware of your ability and limitations in strength and range of movement.
- o **DO NOT** attach to moving surfaces, such as sliding or hinged glass doors.
- o **ALWAYS** check the strength of suction before use.
- o Suction Bathroom Grab Rail will lose suction over time and should be re-set weekly.
- o **DO NOT** attach over tile grout lines, embossed tiles, or uneven surfaces.

ATTACHING THE SUCTION GRAB BAR:

1. Suction Bathroom Grab Rail is designed for use only on smooth and non-porous surfaces, such as tiles, glass, porcelain or fibreglass. **DO NOT USE** on plaster board, wood, painted or papered surfaces.
2. **DO NOT** attach to tiled surfaces where grout lines are spaced apart by 10cm or less. The suction **WILL NOT** work over grout lines.
3. Check the condition of the surface before positioning the Suction Bathroom Grab Rail. Support surface should be free of defects and be in sound condition.
4. Clean surfaces thoroughly and allow to completely dry before attaching the Suction Bathroom Grab Rail.
5. **NEVER** attach to surfaces that have residual soap scum or mould.
6. **NEVER** mount in a position where Suction Grab Rail is in direct sunlight.

Always check the condition of the suction cups. **DO NOT** use if damaged.

INSTALLATION:

1. With the 3 levers raised, place suction cups so they are firmly and completely on a smooth surface. **DO NOT** attach to mosaic tiles or over grout lines.
2. Hold and push Suction Bathroom Grab Rail firmly against the surface, then flip the levers closed. You should feel resistance as the suction tightens against the surface.
3. Always check attachments before each use, and regularly clean the suction cups with a non-abrasive alcohol based bathroom cleaner.
4. To remove, lift all the levers to disengage suction. If suction remains, lift one edge with a finger. **DO NOT** use a sharp edge as this will impair suction capacity and future use.



Green Light Indicates Locked Position



Red Light Indicates Unlocked Position



Sturdy Locking Action